User Manual

530 Chair

Strength in patient care.™

U.S. & Canada (888) 520-4998
International (801) 875-4998
www.mti.net

APPLICABLE MODEL(S).....	IV
REVISION HISTORY TABLE.....	IV
GENERAL INFORMATION.....	1
Purpose of this Manual.....	1
Intended Use.....	1
Symbols and Warnings.....	2
Symboles et Avertissements.....	2
Safety Instructions.....	3
Electromagnetic Interference.....	4
Electrical Requirements.....	4
Contraindications.....	5
Transportation and Storage Conditions.....	5
Disposal of Equipment.....	5
EQUIPMENT FEATURES.....	6
INSTALLATION.....	7
Required Tools.....	7
Explanation of Instructions.....	7
Unpacking & Set-up.....	8
OPERATION.....	9
Operating Conditions.....	9
Controls.....	10
Headrest Operation.....	11
Debris Tray Extension.....	11
Slide Back Floating Arms.....	12
Electrical Lockout.....	12
Mobile Base Operation.....	13
Swivel Base Operation.....	14
LIST OF ACCESSORIES.....	15
MAINTENANCE.....	19
Cleaning and Disinfection.....	20
Preventative Maintenance.....	20
Fuse Replacement Instructions.....	20
Scuff Cover Replacement Instructions.....	21
Calls for Service.....	21
LIMITED WARRANTY.....	21
TROUBLESHOOTING GUIDE.....	22
SPECIFICATIONS.....	24
NOTES.....	25

APPLICABLE MODEL(S)

This manual is for the following part numbers and revisions:

PART NUMBER	DESCRIPTION	EFFECTIVE S/N
5300005	530H Tapered Back Procedure Chair/Table, 115V	080529026 - Current
5300006	530H Tapered Back Procedure Chair/Table, 230V	080529026 - Current
5300009	S530H Square Back Procedure Chair/Table, 115V	080529026 - Current
5300010	S530H Square Back Procedure Chair/Table, 230V	080529026 - Current

REVISION HISTORY TABLE

DATE	REVISION	CO#/ECO#	DETAILS
07/15/2003	ORG	N/A	Initial Release
11/21/2014	A	CO-0060	Update manual to latest revision of chairs with floating arms & new manual format.

User Manual

MTI 530 Chair

GENERAL INFORMATION

Purpose of this Manual

This manual covers the safety, installation, instructions for use, and maintenance of this equipment. It is intended that it be used by the service professional installing the equipment and by the medical professionals who have been trained to use it. No repair information is included with this manual because no operator repairs are intended.

Intended Use

This equipment is intended to be used as a chair/table for general examinations and procedures performed by medical professionals.

PRODUCT INFORMATION

Date of Purchase:_____

Model Number:_____

Serial Number:_____

(see "EQUIPMENT FEATURES" on page 6 for location of serial number label)

Symbols and Warnings

This manual contains Notices, Cautions, and Warnings, which are meant to call attention to particular parts or sections of this manual. Since safety is our first concern, this manual should be read by all personnel who will be operating this medical equipment. Failure to comply with the following warnings and cautions will void the warranty.

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WARNINGS/AVERTISSEMENTS

Are used to call attention to a condition, practice, or procedure which could cause personal injury if not followed correctly. Please become familiar with all warnings and observe them at all times.

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CAUTIONS/PRECAUTIONS

Are used to call attention to a condition, practice, or procedure which could cause product damage if not used correctly.

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NOTICES/AVIS

Are used to call attention to helpful information regarding a condition, practice, or procedure.

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Safety Instructions

WARNING/AVERTISSEMENT

Read this manual before installation and use.

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WARNING/AVERTISSEMENT

No modification of this equipment is allowed.

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WARNING/AVERTISSEMENT

Do not operate equipment in the presence of flammable anesthetics.

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WARNING/AVERTISSEMENT

When the equipment is in motion from a programmable position or auto return, injury may occur if the equipment is not attended to until it comes to a complete stop.

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WARNING/AVERTISSEMENT

Verify equipment, patient, and other objects are in a safe condition before operating the equipment.

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WARNING/AVERTISSEMENT

Verify controls, cords, and other parts of the equipment do not create a tripping or other safety hazard for operator or patient.

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WARNING/AVERTISSEMENT

Lockout switch should be turned to the off position to prevent unintended movement during use of the equipment.

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⚠ WARNING/AVERTISSEMENT

Backrest, seatrest, footrest, armrest, headrest, and other parts of the equipment should only be used for their intended purpose. MTI is not responsible for use of components outside of their intended use.

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⚠ WARNING/AVERTISSEMENT

This product contains chemicals known to the State of California to cause cancer and/or birth defects or other reproductive harm. See <http://www.mti.net/FAQs.aspx> for details on California Proposition 65.

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Electromagnetic Interference

This equipment is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this equipment, do one or more of the following:

1. Plug the equipment into an isolated circuit.
2. Remove the interfering device from the room.
3. Increase the separation between the equipment and the interfering device.
4. Contact MTI if the electromagnetic interference persists.

Electrical Requirements**⚠ WARNING/AVERTISSEMENT**

To avoid risk of electrical shock, this equipment must be connected to a supply mains with protective earth.

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⚠ WARNING/AVERTISSEMENT

Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked "Hospital Only" or "Hospital Grade".

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⚠ WARNING/AVERTISSEMENT

When using endocardial catheters or high frequency surgical/procedure devices, use non-conductive material to insulate the patient from metal portions of the equipment. Failure to comply may result in electric shock or burns to patient.

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⚠ WARNING/AVERTISSEMENT

Connecting electrical equipment to a multiple socket-outlet effectively leads to creating a medical electronic system and can result in a reduced level of safety.

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⚠ WARNING/AVERTISSEMENT

Equipment must be positioned in normal use such that the power cords can be easily accessed to unplug.

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⚠ WARNING/AVERTISSEMENT

Equipment must be connected to an appropriate power source when loss of power source would result in an unacceptable risk.

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⚠ WARNING/AVERTISSEMENT

Do not use a two-three plug adapter to plug the equipment into a two conductor outlet, as grounding reliability is not maintained.

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⚠ WARNING/AVERTISSEMENT

Do not plug this equipment into a low power or auxiliary outlet. This equipment must be connected to a 115 Volt (V) outlet. If this equipment is connected to an insufficient power source, it will cause irreparable damage to this device's electronics.

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⚠ CAUTION/PRÉCAUTION

An appropriately rated surge protector must be used to protect the printed circuit board. Warranty is void if this surge protector is not used.

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❗ NOTICE/AVIS

Removing the power supply cord at the equipment input or wall receptacle will simultaneously isolate both poles of the power supply.

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Contraindications

None.

Transportation and Storage Conditions

	Ambient Temperature	+10°C to +40°C (+50°F to +104°F)
	Relative Humidity	30% to 75% (non-condensing)
	Atmospheric Pressure	700hPa to 1060hPa (0.69atm to 1.05atm)

Disposal of Equipment

At the end of product life, the equipment, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.


EQUIPMENT FEATURES

Description of 530 Chair/Table

This Quad-Power Exam Chair includes power lift, back, tilt and foot with a 650 lb (295 kg) lift capacity with extendable debris tray inside the footrest. The chair is 24" (61.0 cm) wide with a 21" (53.3 cm) minimum height. The chair is used for positioning a patient for exams and procedures.

The 530 chair/table may both be generally referred to as "chair" for convenience in this manual.

530 CHAIR

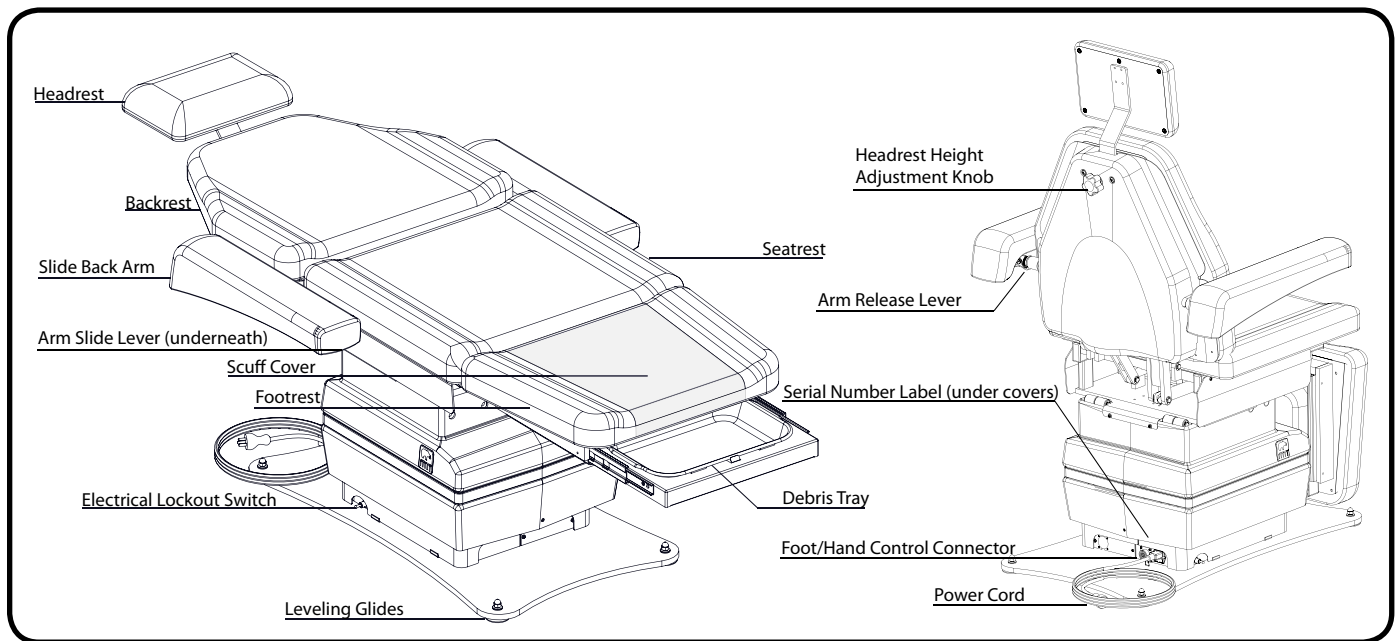INSTALLATION

WARNING/AVERTISSEMENT

This equipment is heavy. Get help to remove it from the shipping pallet. Use proper lifting procedures when lifting to avoid serious back injury.

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WARNING/AVERTISSEMENT

After equipment is installed, verify that the programmable positions are in safe and known positions.

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CAUTION/PRÉCAUTION

To move equipment, lift only from the equipment's main structure.

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NOTICE/AVIS

If this product has been exposed to low temperatures, allow adequate time for it to warm to room temperature before putting it into service.

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
NOTICE/AVIS

Inspect the shipping boxes for visible damage. Record on bill of lading, photograph, and report any damage to the carrier immediately. Request an inspection by the carrier if any concealed damage is noted.

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
Required Tools

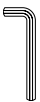	7/16" Wrench 5/8" Wrench		Wire cutters		Utility Knife		Flat Head Screwdriver
	Phillips Screwdriver (#2)		5/64" Allen Wrench		Swivel Base Only: 9/16" Wrench		


Explanation of Instructions

Instructions have steps which are designated by a number. Each step may also have sub-steps designated by a letter. For example, an instruction to operate a feature may have 4 steps (1-4) and each of those steps may have sub-steps (A-B) to be performed before moving on to the next step.


Unpacking & Set-up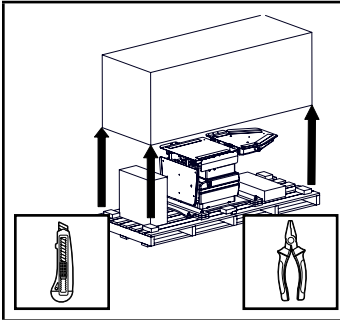


1. Remove shipping straps, shipping box, and other boxes from the pallet.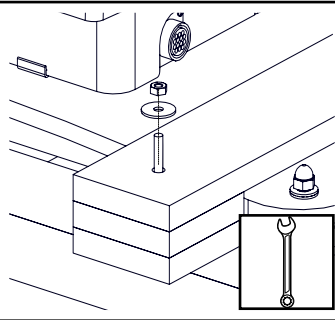


2. Remove (4) shipping bolts with 7/16" end wrench.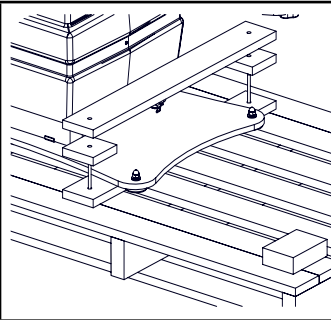


3. Remove braces from the top of the baseplate. Braces will vary for swivel/mobile bases.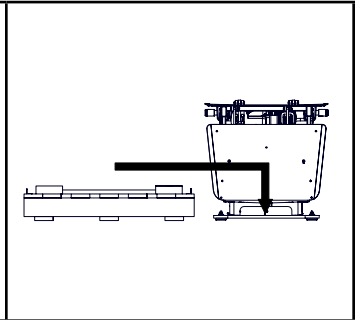


4. Slide chair off the pallet. Lift only from the equipment's main structure. Recycle or re-use pallet.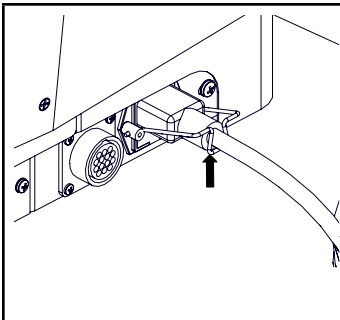


5. Insert power cord at the base of the chair and secure with the clip. Plug the power cord into the provided surge protector. Plug the surge protector into an appropriately marked receptacle.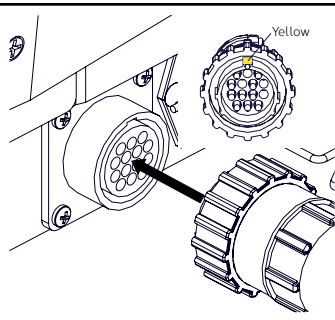


6. Install control(s) by aligning pins and tabs of the male control plug with chair receptacle by positioning the yellow mark on the plug body to the 12 o'clock position relative to the receptacle at the lower rear of the chair base.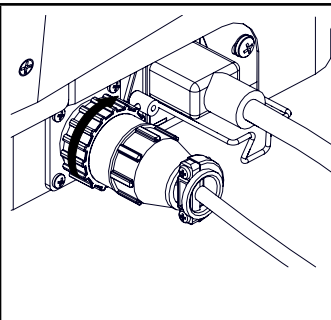


7. Secure the plug by rotating the locking ring clockwise until it engages. Rotate it until it stops. Steps 8 through 9 do not apply to mobile base chairs.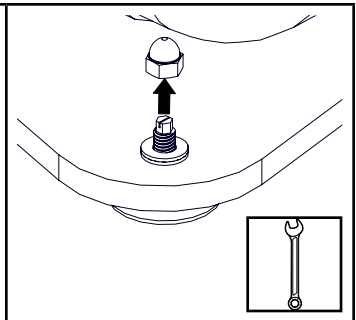


8. If chair base plate needs to be leveled, remove (4) acorn nuts from the top of the glides with a 5/8" wrench. otherwise proceed to step 10.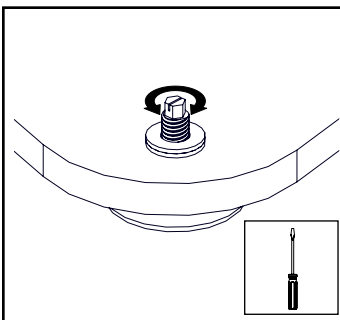


9. Adjust individual glides to level the chair as needed using a screwdriver. Reattach the acorn nuts.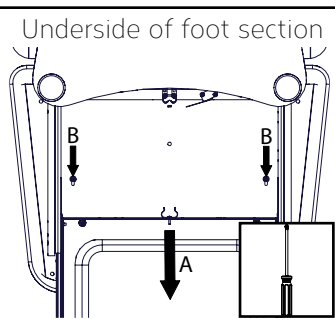


10. A. Raise the foot section and extend the debris tray. B. Align the screws on the foot cushion with the slotted holes in the foot section, slide the cushion in place, and tighten the screws.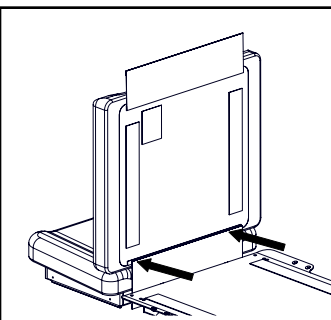

11. Attach the flap on the top of the foot cushion to the Velcro® on the seat cushion. The rear section of the seat cushion has a flap already attached.

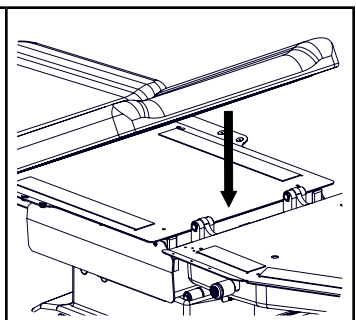12. Align the seat cushion with the seat section of the chair and press into place. Verify the cushion is centered.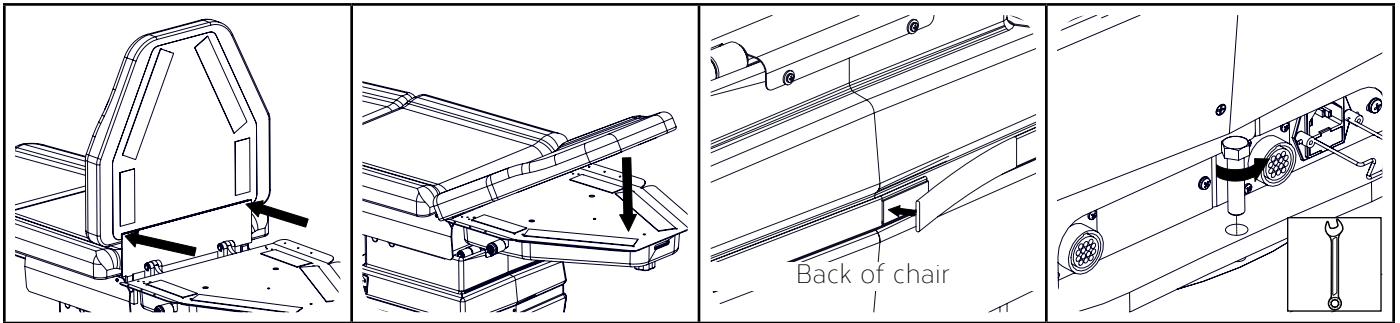

13. Attach the flap on the rear of the seat cushion to the Velcro® on the back cushion.

14. Align the back cushion with the back section of the chair and press into place. Verify the cushion is centered.

15. Wrap the decorative base strap around the plastic base cover and attach Velcro®.

16. Swivel base chairs only. Remove swivel bolt with 9/16" wrench. Keep the bolt for use while the chair is not powered.


OPERATION

Operating Conditions

	Ambient Temperature	+10°C to +40°C (+50°F to +104°F)
	Relative Humidity	30% to 75% (non-condensing)
	Atmospheric Pressure	700hPa to 1060hPa (0.69atm to 1.05atm)

Controls

Foot Control/Hand Control/Hand Switches

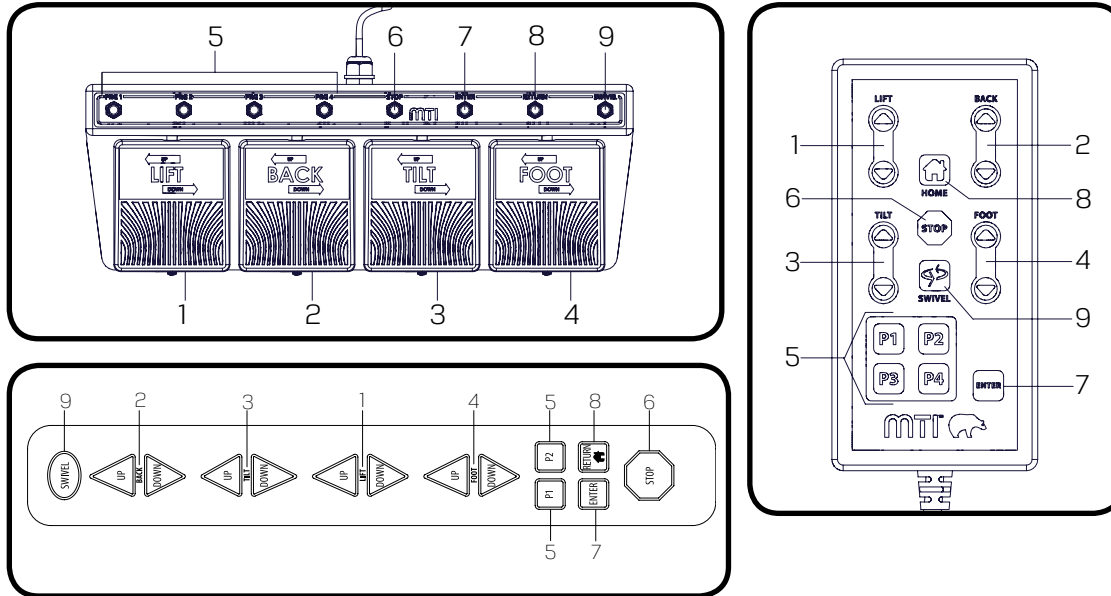ITEM	DESCRIPTION	FUNCTION
1	Lift Pedal/Button	Raises or lowers the chair top by pressing the desired direction
2	Back Pedal/Button	Raises or lowers the backrest by pressing the desired direction
3	Tilt Pedal/Button	Decreases or increases the angle of the seat by pressing the desired direction
4	Foot Pedal/Button	Raises or lowers the footrest by pressing the desired direction
5	Program Buttons (4)	Moves the chair to a user programmed position
6	Stop Button	Stops all movement of chair
7	Enter Button	Press this button followed by a program button to save the current position of the chair as a programmed position.
8	Return Button	Returns the chair to the home position (Lift Down, Tilt Down, Back Up, and Foot Down)
9	Swivel Button	Pressing and releasing the swivel button within two seconds will allow the chair to swivel for six seconds or until the swivel or any other button is pressed. Pressing and holding the swivel button for longer than two seconds will allow the chair to swivel until the button is released. See "Swivel Base Operation" on page 14 for more details.

! NOTICE/AVIS

For additional safety, any button will stop all motions except for the same button that was pressed to initiate movement.

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
! NOTICE/AVIS

Swivel button is present on foot control for swivel base chairs only.

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Headrest Operation

Articulating and Non-Articulating Headrests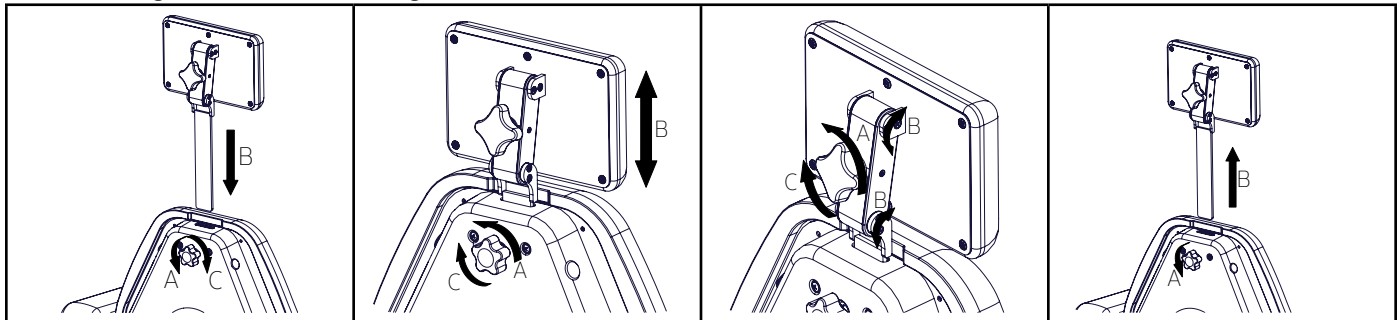

1. To install the headrest:
 A. To loosen, turn the knob on the back of the chair counterclockwise.
 B. Insert the headrest through the slot in the top of the backrest.
 C. To secure, turn the knob clockwise.


2. To adjust the height of the headrest:
 A. To loosen, turn the knob on the back of the chair counterclockwise.
 B. Adjust the height of the headrest.
 C. To secure, turn the knob clockwise.

3. To adjust the angle of the headrest (Articulating only):
 A. To loosen, turn the knob on the back of the headrest counterclockwise.
 B. Adjust the position and angle of the headrest.
 C. To secure, turn the knob clockwise.


4. To remove a headrest:
 A. To loosen, turn the knob on the back of the chair counterclockwise.
 B. Pull the headrest completely out of the back.

Debris Tray Extension

The following instructions illustrate the operation of the extendable debris tray.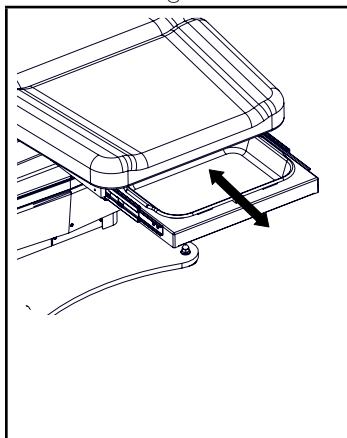

1. To extend the debris tray grasp the front of the tray and pull out.
 When the tray is extended, the movement of the foot section is disabled. To re-enable the movement of the foot section, push in the debris tray completely.
 2. To retract the debris tray, grasp the front of the tray and push it in.

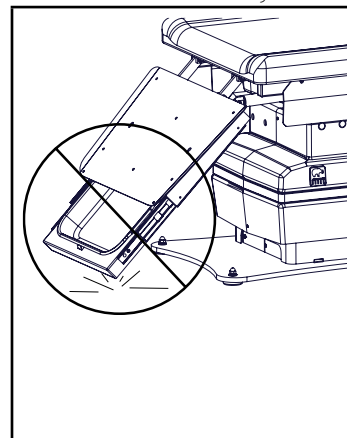WARNING/ **AVERTISSEMENT**

Retract the debris tray before pressing the return or a program button where the footrest may hit the ground.

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
NOTICE/AVIS

The extendable debris tray may come into contact with the floor if the foot section and debris tray are in a position where adjusting the tilt or lift would cause the debris tray to contact the floor.

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Slide Back Floating Arms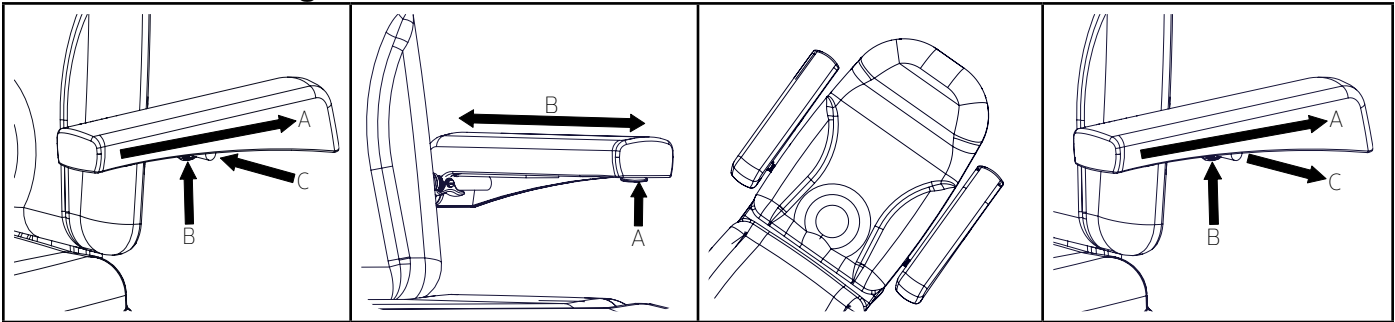

1. To attach the arm:

A. Press the lever on the front of the arm and slide the arm backwards.

B. Press the lever on the spline of the arm.

C. Insert the male spline into the female spline on the side of the chair.

2. To slide the arm backwards or forwards:

A. Press the lever on the front of the arm.

B. Slide the arm.

! The arm will lock in place in the forward and backward positions.

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3. For prone patient positions: Swap the arms on each side and insert them angled down slightly or parallel to the backrest.

4. To remove the arm:

A. Slide the arm backward.

B. Press the lever on the spline arm.

C. Pull the arm straight out.


! NOTICE/AVIS

Positioning the arm in the back position will make it easier to attach or remove the arm.

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
Electrical Lockout

The electrical lockout cuts power to all motors to eliminate possible EMI (electromagnetic interference) and RFI (radio frequency interference). This switch also resets the circuit board.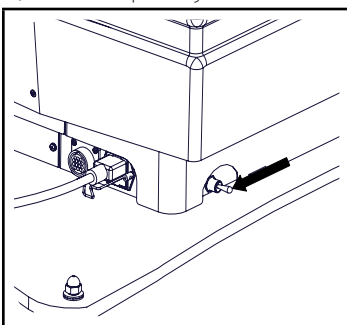

1. To lock out control of the chair:

A. Switch the toggle switch to the off position (pointing toward the backrest of the chair).

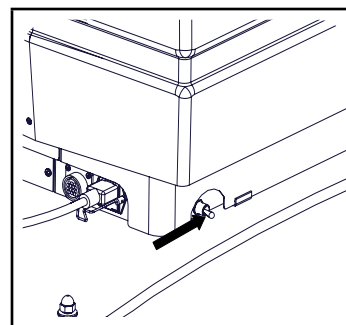2. To enable control of the chair:

A. Switch the toggle switch to the on position (pointing toward the footrest of the chair).

! NOTICE/AVIS

The toggle switch will not lock out non-powered motion such as footrest extension.

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Mobile Base Operation

The mobile base is a factory installed option for the 530 Model chairs.

The two casters on the patient's right side and the rear caster on the patient's left side of the chair are Total Lock Casters. The front caster on the patient's left side is the Steering Caster.

In the FREE or STEER position, the Total Lock Casters will roll and swivel. In the LOCK position, the Total Lock Casters will not roll or swivel.


In the STEER position, the Steering Caster will roll but not swivel. In the FREE position, the Steering Caster will swivel and roll. In the LOCK position, the Steering Caster will not roll or swivel.

There are three modes for the offset locking bars. See image on right.

STEERING MODE: When the left-hand locking bar is in the STEER position and the right-hand locking bar is in the FREE or STEER position; the (3) Total Lock Casters will roll and swivel freely and the (1) Steering Caster will roll but will not swivel. This mode is used for transport as the Steering Caster acts as a steering wheel to aid in easy transport.

FREE MODE: When both offset locking bars are in the FREE position; all (4) casters will roll and swivel freely. This position is used when you need to pull the chair away from an object in a perpendicular direction, or turn the chair with a zero radius.


LOCKED MODE: When both offset locking bars are in the LOCK position; all (4) casters will not roll or swivel. This position is used to prevent unwanted movement of the chair.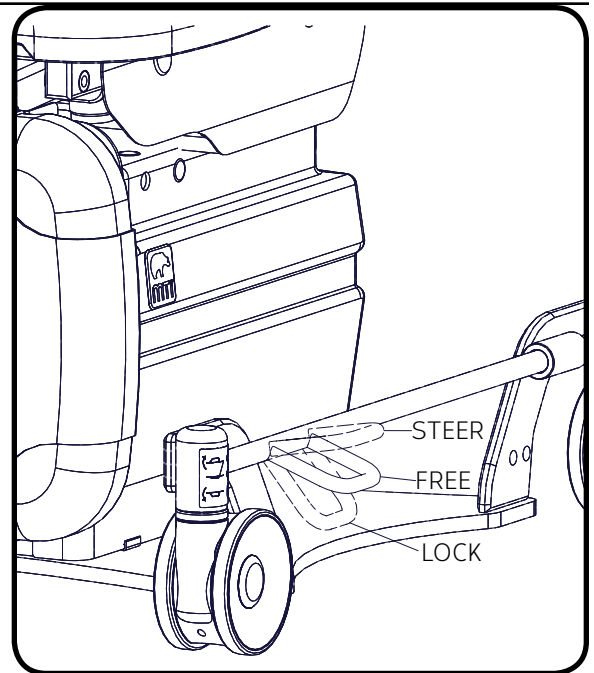

Swivel Base Operation

The swivel base is a factory installed option for the 530 Model chairs. See also "Controls" on page 10 of this manual. Swivel mode is the condition when the swivel lock is unlocked and the chair can swivel freely.

There are two methods to operate the swivel on the 530 Model chairs. Follow the step(s) below for the desired method.

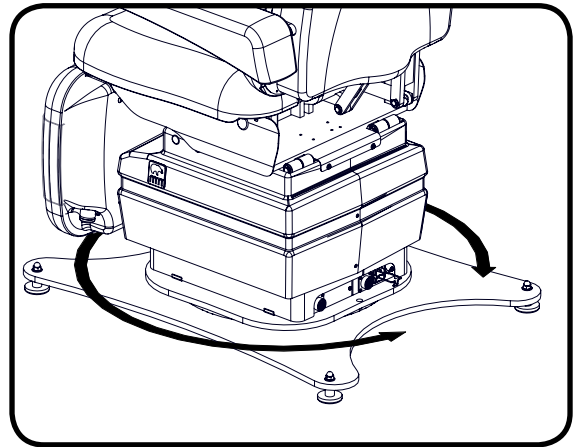Method 1: Auto engage mode is a swivel mode which will unlock the swivel for 6 seconds and automatically engage the lock after 6 seconds.

1. Press the swivel button on the control and release it within two seconds.
2. Swivel enters swivel mode for 6 seconds.
3. Swivel the chair to the desired location.
4. After 6 seconds the swivel lock will automatically engage.
5. If the chair is in swivel mode and the swivel button is pressed again, the swivel lock immediately engages.
6. If the chair is in swivel mode and any other command button is pressed, the swivel lock immediately engages.

Method 2: Press and hold mode is a swivel mode where the swivel will unlock as long as you hold down the swivel button on the control. Once you release the swivel button, the swivel lock will engage.

1. Press and hold swivel button on the control for longer than two seconds.
2. The chair enters swivel mode until the swivel button is released.
3. Swivel the chair to the desired location.
4. The chair stays in swivel mode as long as the swivel button is held down and immediately engages swivel lock upon button release.

! NOTICE/AVIS

The swivel will lock if the swivel is unlocked and another button is pressed.

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! NOTICE/AVIS

The swivel will unlock if the chair loses power.

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LIST OF ACCESSORIES

DESCRIPTION	USE	RESTRICTIONS
Accessory Rail Mounting Bracket	Bracket to mount various devices such as hyfrecators and lights.	Requires Base Rails. Not compatible with Bed Rails.
Arm, Deluxe 8-way 18" IV/Surgery (Floating or seat mounted)	Provides an articulating surface for patient's arms.	Seat mounted requires seat rails.
Arm, Standard 8-way, 18" IV/Surgery (Floating or seat mounted)	Provides an articulating surface for patient's arms.	Seat mounted requires seat rails.
Arm, Deluxe 8-way Back Rail Mounted 24" IV/Surgery	Provides an articulating surface for patient's arms.	Requires Back Rails.
Arm, Standard 8-way Back Rail Mounted 24" IV/Surgery	Provides an articulating surface for patient's arms.	Requires Back Rails.
Arms, Slide Back Floating (set)	Used by patient to rest their arms during exams/procedures.	
Arm, Standard 4-way Removable Patient Arm (Floating and Seat Mtd)	Used by patient to rest their arms during exams/procedures.	Seat mounted requires seat rails.
Base Mount 1/2" Post	Provides an additional base mount to facilitate moving 1/2" posts to other side of chair.	
Bed Rails, Swing Up/Down (Floating)	Provides a vertical surface to secure patient.	
Bed Rails, Swing Up/Down (Mounted to Slide Back Floating Arms)	Provides a vertical surface to secure patient.	Requires Slide Back Floating Arms.
Body Strap	Used for securing a patient to the chair during procedures.	
Casters with individual locks	Provides field installable casters to make the chair mobile.	Not available on mobile or swivel base chairs.
Corded Hand Control w/Mount	Provides an additional control for the chair and mount for hanging the hand control.	
Hand Control Caddy, Flexible	Used to hold a hand control.	Requires Seat or Base Rails.
Hand/Arm Surgery Table	Provides an articulating surface for patient's arms.	Requires Back Rails.
Hand/Forearm Procedure Table (Floating or seat mounted)	Provides an articulating surface for patient's arms.	Seat mounted requires seat rails.
Hand switch option	Provides hand switches on back covers.	Square or Tapered back Not available with back rails.
Headrest, Articulating Oval, Deep Oval, Prosthetic, Round, Horseshoe/Donut	Provides a dual articulating surface with various pillow shapes/sizes to support patient's head.	
Headrest, Non-Articulating Oval, Rectangular	Provides a surface with oval or rectangular pillow shapes which do not articulate.	

LIST OF ACCESSORIES

DESCRIPTION	USE	RESTRICTIONS
IV Rod, Base Mounted	Provides an adjustable IV rod.	
IV Rod and Holder	Provides an adjustable IV rod mounted to a 1/2" Post	Requires 1/2" Post.
IV Rod and Holder	Provides an adjustable IV rod mounted to Base Rails.	Requires Base Rails.
IV Rod and Holder	Provides an adjustable IV rod mounted to 2" post (external or internal).	Requires 2" Post.
IV Rod and Holder	Provides an adjustable IV rod mounted to a 1" post.	Requires 1" Post.
Leveling Glides, 2"	Provides taller leveling glides than standard.	Not available on mobile or swivel base chairs.
Light, Lumerus LED Exam	Provides lighting during patient procedures.	Requires Base Mounted Post or Base Mounted Mayo Tray w/Light Adapter.
Light, MI 500 LED Exam	Provides lighting during patient procedures.	Requires 2" Post.
Light, Operatory	Provides lighting during patient procedures.	Requires 2" Post.
Light, Surgery, MI 1000 LED	Provides lighting during patient procedures.	Requires 2" Post.
Mayo Tray with Light Adapter, Base Mounted (RH/LH)	Combines a mayo tray and light on one mount.	Must be mounted on opposite side of any Base Mounted Accessory.
Mayo Tray, Base Mounted (RH/LH)	Provides a tray for the placement of small instruments and other items.	Must be mounted on opposite side of any Base Mounted Accessory.
Mayo Tray, Flexible	Provides a tray for the placement of small instruments and other items.	Requires 2" Post.
Mayo Tray, 1" Post Mounted	Provides a 1" post with a tray for the placement of small instruments and other items.	
Mayo Tray, 1" Post Mounted with Light mount	Provides a 1" post with a tray for the placement of small instruments and other items and a mount for a 42" Lumerus Light.	
Mobile Caster Base	Provides field installable base with casters to make chair mobile and caster jacks.	Not available with low boy mobile base or swivel base chairs.
Mobile Caster Base, Low Boy	Changes a standard chair into a mobile base.	Factory installed
Mobile Caster Cart	Provides field installable casters to make the chair mobile.	Not available on low boy mobile base or swivel base chairs.

LIST OF ACCESSORIES

DESCRIPTION	USE	RESTRICTIONS
Monitor Tray, Post Mounted	Mounts a patient monitor for viewing during procedures.	Requires Base Mounted Post.
Monitor Tray, 2" Post Mounted	Mounts a patient monitor for viewing during procedures.	Requires 2" Post.
Oxygen Tube, Flexible	Used to supply oxygen to the patient and/or a flexible drape support.	
Oxygen Tube/Drape Support, Rigid	Used to supply oxygen to the patient and/or a rigid drape support.	Requires Back Rails.
Outlet Kit, Duplex, 115V	Provides power for 115V accessories.	Left hand version only Not available for 230V Chairs.
Paper Roll Holder	Used for mounting a paper roll to the back of the chair.	18" (45.7 cm)
Post, 1"	Provides a post for mounting a 42" Lumerus light and other accessories.	
Post, 2"	Used to mount surgery lights, and other accessories.	
Post, Base Mounted (RH/LH)	Used in combination with a Monitor Tray or Light.	Must be mounted on opposite side of any Base mounted accessory.
Power Cord with 90 degree plug, 3'	Provides a 3' power cord with a 90 degree plug.	Not available on mobile or swivel base chairs.
Rail Clamp	Provides a clamp that can lock onto 1-1/8" x 3/8" size rails and accepts 5/8" post.	
Rails, Back (set)	Provides standard hospital rails to the back of the chair for mounting accessories.	
Rails, Base (set)	Provides rails for mounting an IV pole or accessory bracket.	Cannot be used with Base Mounted Mayo Tray or Base Mounted post.
Rails, Seat (set)	Provides standard hospital rails to the seat of the chair for mounting accessories.	
Restraint Strap, Leg	Provides a strap to secure the patient's legs.	
Restraint Strap, Body with snaps or Velcro®	Provides a strap to secure the patient's body.	Strap with snaps mounts to seat frame. Strap with Velcro® requires seat rails and fastens around seat rails.
Shoe Tray, Chair Mounted	Provides a place for a patient's shoes during procedures.	

LIST OF ACCESSORIES

DESCRIPTION	USE	RESTRICTIONS
Shoe Tray, Post Mounted	Provides a place for a patient's shoes during procedures.	Requires Post-mounted Mayo Tray or Post-mounted Light.
Slant Mount, Foot Control	Provides a slanted surface to mount a foot control at the rear of the chair.	
Steering Handles, Back Rail Mounted Removable	Used for moving the chair.	Requires Back Rails.
Stirrups (LH/RH)	Used to position patients' legs during procedures.	Requires Seat Rails.
Swivel Base	Allows chair to swivel.	Factory Installed. Not available with any mobile base or with a power cord with 90 degree plug.
Syringe Bracket Holder	Provides a dovetail mount to attach a syringe holder to an IV/Surgery Arm.	
Visual Block Curtain, Chair Mounted	A curtain to shield procedures from the patient's view.	
Visual Block Curtain, Post Mounted	A curtain to shield procedures from the patient's view.	Requires Post-mounted Mayo Tray or Post-mounted Light.
Visual Block and Shoe Tray, Chair Mounted	A curtain to shield procedures from the patient's view combined with a tray for shoes.	
Visual Block and Shoe Tray, Post Mounted	A curtain to shield procedures from the patient's view combined with a tray for shoes.	Requires Post-mounted Mayo Tray or Post-mounted Light.

MAINTENANCE

WARNING/AVERTISSEMENT

There are no user serviceable parts inside. Refer all servicing to a qualified service technician.

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WARNING/AVERTISSEMENT

After equipment is serviced, verify programmable positions are safe and known positions.

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WARNING/AVERTISSEMENT

The Control of Hazardous Energy (lockout/tagout) §1910.147 procedure must be followed to disable equipment, preventing the release of hazardous energy while performing servicing and maintenance activities.

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CAUTION/PRÉCAUTION

This equipment may be damaged by solvents and dyes. Immediately remove any fluids spilled on the equipment.

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CAUTION/PRÉCAUTION

Avoid contact with sharp or pointed objects. Do not place hard surface objects on the upholstery surfaces as temporary indentations or impressions in the upholstery cover will result. Prolonged contact with a hard or irregular shaped object may make the impression semi-permanent.

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CAUTION/PRÉCAUTION

Any sprayed disinfectant or cleaning solution should be wiped up immediately after application and not let to stand. A small sample area should be tested before any large application of any cleaner/disinfectant.

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CAUTION/PRÉCAUTION

Avoid prolonged direct sunlight exposure to preclude color change of the plastic.

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NOTICE/AVIS

MTI recommends cleaning the equipment frequently.

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Cleaning and Disinfection

Upholstery

Upholstery should be cleaned weekly with a mild liquid soap and water mixture. Afterwards, wipe the upholstery with a clean damp cloth.

MTI's standard policy on disinfecting MTI medical equipment is to use a 10% sodium hypochlorite (bleach) and 90% water solution. If stronger or other disinfecting and cleaning solutions are used, the surface is at risk to be damaged or possibly cause discoloring, cracking, or other types of damage to the surface. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

MTI understands that many customers prefer to use a pre-mixed disinfecting solution in a spray or wipe. It is MTI's recommendation not to use sprays because over time the overspray will cause damage to the surfaces if not immediately wiped up. MTI recommends hand towels or wipes to avoid the possibility of overspray. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

See "MTI Cleaning Solution Recommendation Letter" for more details on disinfecting solutions which have been tested on MTI's products. It is available for download at <http://www.mti.net/downloads.aspx> in the Product Support Bulletins folder, found under the Resources tab on our website. In the U.S. & Canada, call (888) 520-4998, or International, call (801) 875-4998 to request a copy.

Painted and Plastic Surfaces

All painted metal parts have been electrostatically painted and the paint is baked on for durability. These surfaces should be wiped down with a quality grade cleaner and a soft cloth or a damp cloth with mild soap once a week.

Preventative Maintenance

Periodically inspect the following areas:

1. All fasteners and pins should be in place and tightened securely.
2. Lubricate pins and moving joints as needed with lithium grease.
3. Power cord should be free of cuts or other visible damage.
4. All mechanical functions should operate properly.

Have an authorized service technician inspect your equipment once a year.

Fuse Replacement Instructions

1. Unplug the chair.
2. Open the fuse drawer below the female connection for the power cord.
3. Remove the burnt fuse(s). (Needle nose pliers or hemostats may be needed.)
4. Insert new fuse(s) with same type of fuse removed in step 3 (See figure 1 below). See "SPECIFICATIONS" on page 24 for fuse type.
5. Close the fuse drawer.
6. Reinsert the power cord into the base of the chair and secure it with the retainer clip.

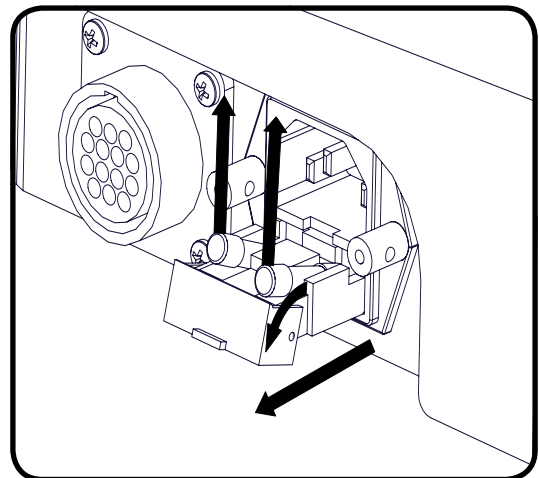
Figure 1

Scuff Cover Replacement Instructions

1. Remove the foot cushion from the chair by loosening the two screws on the underside of the foot using a Phillips screwdriver. (See also "Unpacking & Set-up" on page 8.)
2. Remove the scuff cover from the foot section (attached with Velcro®).
3. Place the replacement scuff cover on a flat surface.
4. Place the foot cushion on top of the scuff cover, centered left to right. (See Figure 1.)
5. Attach the bottom edge of the Velcro® on the scuff cover with the Velcro® on the foot cushion. Stretch the scuff cover over the bottom edge.
6. Slide the foot cushion on the scuff cover to tighten.
7. Pull the top edge of the scuff cover over the top of the cushion and secure the Velcro®.
8. Reattach the foot cushion to the chair. (See also "Unpacking & Set-up" on page 8 for installation instructions.)

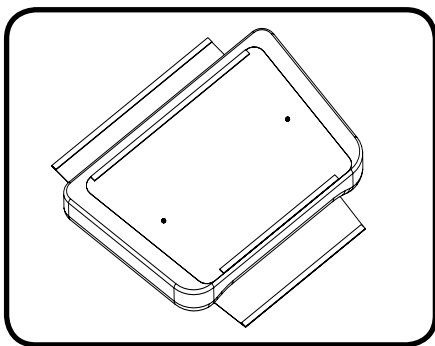
Figure 1

Calls for Service

Equipment's model and serial number information are required when calling for service.

Serial number is indicated by the **SN** symbol located on the serial number tag.

If service is required, contact MTI

U.S. & Canada 888-520-4998

International 801-875-4998

Fax (801) 952-0548

customer.service@mti.net

Monday-Friday 8:00 am - 5:00 pm (MST)

www.mti.net

LIMITED WARRANTY

Medical Technology Industries, Inc. (hereinafter referred to as MTI), shall repair or replace products of its manufacture, which prove to be defective in material and/or workmanship for a period of twelve (12) months from the date of shipment to the customer (the "Warranty Period"). MTI will, at its option, provide parts to the customer or repair the defective part at MTI's factory or authorized repair facility. Except as it may otherwise specifically agree in writing, MTI shall not be liable for transportation or labor charges for repairs or adjustments or other work done on any of its products by MTI's authorized dealers or service organizations.

Customer shall notify MTI of any alleged claim or defect within fourteen (14) days from the date customer discovers, or upon reasonable inspection should have discovered, such alleged claim or defect (but in any event before the expiration of the Warranty Period). All repair parts are warranted for Ninety (90) calendar days from the date of shipment to the customer. Warranty is non-cumulative.

This warranty does not apply to any part or product which upon examination by MTI or its authorized agents indicates the product has been repaired or altered in any way by persons other than MTI or its authorized agents, has been misused, abused or used in a manner contrary to any instructions issued by MTI. Any parts or products claimed under this warranty must be properly packaged by customer and returned to MTI's factory with no liability to MTI for the parts, products or transportation charges thereon.

In no event, shall MTI have any liability for consequential, indirect, incidental, special, exemplary, punitive or enhanced damages, lost profits or diminution in value arising out of or relating to a product defect or the repair or replacement of any products manufactured or sold by it.

THE FOREGOING LIMITED WARRANTY IS MADE IN LIEU OF ALL WARRANTIES EXPRESSED OR IMPLIED AND MTI MAKES NO OTHER WARRANTY WHATSOEVER WITH RESPECT TO THE PRODUCTS INCLUDING BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

TROUBLESHOOTING GUIDE

NOTICE/AVIS

For more information on this equipment, including circuit diagrams, parts and service manuals, and other descriptions, please contact MTI.

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#	SYMPTOM	PROBLEM CAUSE	SOLUTION
1	Chair will not function.	Chair power cord is not plugged in.	Verify power cord is securely connected at both ends.
2	Chair will not function.	Facility power receptacle problem.	Check facility's power receptacle with a known working device of same voltage specifications.
3	Chair will not function.	Facility power receptacle problem.	Check facility circuit breaker to verify it is not tripped.
4	Chair will not function.	Facility supply voltage is incorrect voltage.	Verify voltage is per specifications on chair serial number/product tag.
5	Chair will not function.	Electrical lockout switch is off.	Flip the Electrical Lockout Switch. See "Electrical Lockout" on page 12.
6	Chair will not function.	Fuses have blown.	Verify fuses are not burnt. Replace fuse(s) if necessary. See "Fuse Replacement Instructions" on page 20.
7	The circuit board has to be reset often.	Possible faulty circuit board.	Contact Customer Service.
8	Chair will not function or foot section does not move.	The debris tray is not fully inserted.	Insert the debris tray.
9	Chair will not function.	Circuit board isn't functioning correctly.	Contact Customer Service.
10	Chair will not function or a function is erratic.	Controls are not securely connected.	Verify each control is securely connected to the chair. See "Unpacking & Set-up" on page 8 for instructions how to securely connect controls.
11	Lift function is slower than usual and only goes up about halfway.	One of the lifting column motors is not functioning or circuit board isn't functioning correctly.	Contact Customer Service.

TROUBLESHOOTING GUIDE

NOTICE/AVIS

For more information on this equipment, including circuit diagrams, parts and service manuals, and other descriptions, please contact MTI.

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#	SYMPTOM	PROBLEM CAUSE	SOLUTION
12	Back, Tilt, Lift or Foot functions will not function.	Circuit board has disabled motor(s). The controller may disable individual motor(s) to prevent motor damage in the event that a failure in the circuit board is detected or unusual motor operation has occurred.	Reset circuit board. For chairs with a circuit board revision AG (June 2014) and after, flip the electrical lockout switch to off position, then to the on position. See "Electrical Lockout" on page 12. For chairs with a circuit board revision AD and prior, unplug the chair from power and wait one minute and plug back in.
13	Back, Tilt, Lift or Foot functions will not function.	Motor is not functioning or circuit board isn't functioning correctly.	Contact Customer Service.
14	The debris tray is retracted and the footrest will not move.	Broken foot section switch.	Contact Customer Service.
15	Swivel will not lock.	Wiring, switch, and/or circuit board is not functioning correctly.	Contact Customer Service.

SPECIFICATIONS

DESCRIPTION	SPECIFICATION
Lifting Capacity	650 lbs (295.45 kg)
Weight of Chair*	364 lbs. (166 kg)
Table Top Range	21" (53.3 cm) to 37" (94.0 cm)
Back Section Range	0° (horizontal) to + 85° ± 5°
Table Top Tilt Range	0° to 30°
Foot Section Range	0° (horizontal) to + 90° ± 5°
Debris Tray Extension	10.5" (26.7 cm)
Foot Control	Rated Drip Proof
Hand Control	Rated Drip Proof
General Dimensions (with headrest)	24" (61 cm) wide x 70.5" (179.1 cm) long (excludes debris tray)
Power Cord	Plugs: U.S.A. standard 3 prong and IEC 60320-C13.
Electrical Ratings:	115 VAC ± 10%, 4 amps, 60 HZ 230 VAC ± 10%, 2 amps, 50 HZ
Fuse Replacement	PC Board: 32 V, 20 A, Fast Acting, 6.3 mm x 32 mm At power cord inlet: 250 V, 4 Amp, Slo-Blow, on 115 V, 5mm x 20mm 250 V, 2 Amp, Slo-Blow, on 230 V, 5mm x 20mm
Duty Cycle	Intermittent Operation, 10% Duty Cycle Example: (1) minute on (9) minutes off
Classification	Class 1, Type B Equipment, Ordinary Equipment, Intermittent Operation
Listing/Approvals	ETL, UL 60601-1, 1st Edition, CAN/CSA-C22.2 No. 601.1-M90, CE
* Weight and dimensions may vary depending on accessories purchased and international shipment weights and dimensions may vary.	

NOTES

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

[illegible]

Strength in patient care.™

3655 W. Ninigret Drive
Salt Lake City, Utah, 84104-6572
(800) 924-4655 • (801) 875-4999 • Fax (801) 952-0548
www.mti.net

© MTI, Inc. All rights reserved.
MTI reserves the right to make any
product changes without notice.

073007 Rev A